

COUNTERFIT™

QUICK SET

MULTI-PURPOSE REPLICATION SILICONE



IMPRESSIONS

Every dental office uses alginate for various techniques when a quick impression is required. While the use of alginate is routine, so too are its inherent problems and shortcomings. It is well known that alginate is not stable due to shrinkage from dehydration, and that it requires immediate pouring, but alginate also has many other shortcomings, including; susceptibility to improper mixing, and inaccurate powder-water ratios, as well as messy clean up chairside, in the lab, and of the patient. Alginate also produces less than ideal accuracy on important cases like temporary matrix fabrication and opposing models for crown and bridge applications. Additionally, stone models derived from alginate will not fit vinyl poly siloxane bite registrations. Simply put, alginate is unpredictable.

COUNTER-FIT™ QUICK SET will provide exact replication, superior to alginate, for all the procedures for which you would normally use alginate. **COUNTER-FIT** sets in about the same time as alginate, is thixotropic so it won't induce gagging, and it provides outstanding replication of detail. What about stability? **COUNTER-FIT** remains stable for years, is easily disinfected, and can be poured immediately (or whenever convenient) as well as multiple times – something you just could not do with alginate.

APPLICATIONS


Counter-Fit Quick Set is ideally suited for any application requiring occlusal accuracy.

- Opposing arch impressions
- Bleaching tray & athletic mouthguard fabrication
- Study models
- Orthodontic models, appliances, and retainers
- Matrix material for long span provisional crown and bridge restorations
- TMJ splints
- All alginate-type impressions


MAXIMUM WORKING TIME:	1:00 min.
MINIMUM INTRAORAL SET TIME:	1:30 min.
TOTAL SET TIME:	2:30 min.




COUNTER-FIT DERIVED MODELS
Ideal bite registration fit

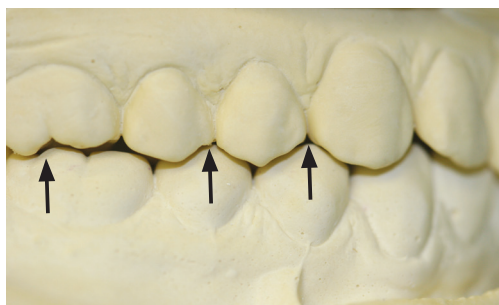

COUNTER-FIT DERIVED MODELS
Closed bite

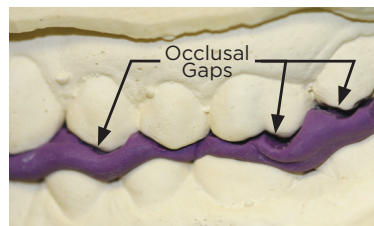

ALGinate DERIVED MODELS
Poor fit

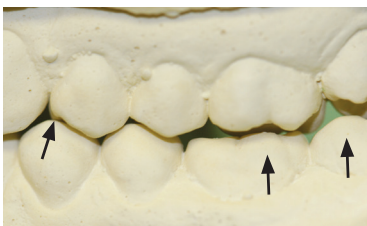

ALGinate DERIVED MODELS
Open bite



COUNTER-FIT MegaMix Cartridge

Dentistry and photography courtesy of Dr. Len Boksman and Gregg Tousignant.

CLINICIAN'S CHOICE®
DENTAL PRODUCTS INC.

www.clinicianschoice.com


COUNTERFITTM

QUICK SET


MULTI-PURPOSE REPLICATION SILICONE




CLINICAL TECHNIQUE

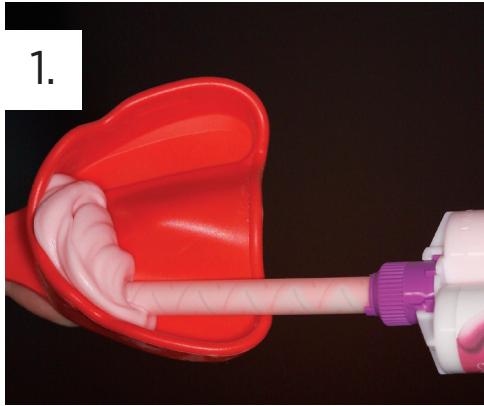

Counter-Fit Quick Set loaded directly into HeatWave tray.

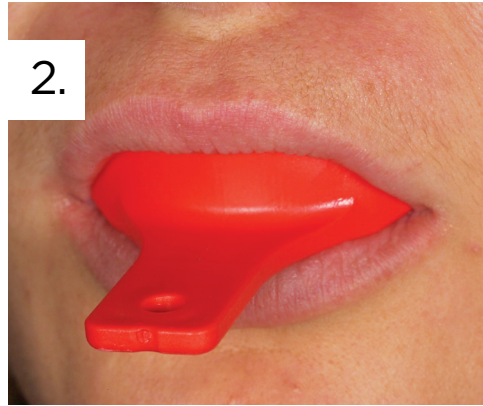

Counter-Fit Quick Set impression of upper arch.

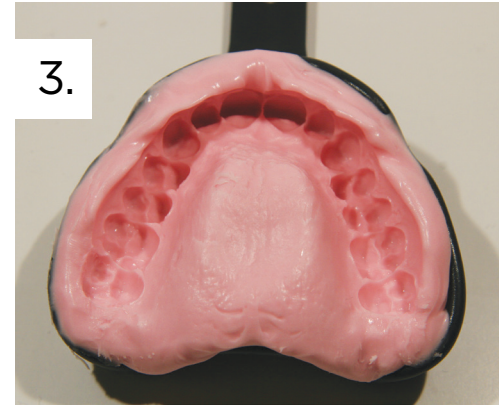

Superb replication of detail in the Counter-Fit Quick Set impression which can be poured up whenever convenient. Counter-Fit impressions can be poured multiple times.

Dentistry and photography courtesy of Dr. Todd Snyder

CLINICIAN'S CHOICE[®]
DENTAL PRODUCTS INC.

www.clinicianschoice.com