

**INTENDED USE:** Dental finishing and polishing points intended as a medical device for the purpose of final polishing of all composite and ceramic restorations.

## **TECHNIQUE**

### **STEP 1:** D-FINE Double Diamond Primary Polishers (purple)

1. Use a finishing diamond/carbide to contour the composite surface.
2. Select the appropriate D-FINE Double Diamond Primary Polisher.
3. Using a wiping motion, at 7,500-10,000 rpm, and with light to moderate pressure and water spray, apply the primary polisher to the restorative surface. A smooth silky surface will appear in approximately 30 seconds.

### **STEP 2:** D-FINE Double Diamond Final High Shine Polishers (orange)

1. Select the appropriate D-FINE Double Diamond Final High Shine Polisher.
2. Again, with light to moderate pressure and water spray, apply the Final High Shine Polisher to the restorative surface with a wiping motion, until a high gloss finish is achieved.

**NOTE:** Failure to follow this technique will lead to premature degradation of the polishers.

## **RECOMMENDED STERILIZATION**

D-FINE Double Diamond polishers should be steam autoclaved only up to 275°F/135°C. Product is not sold in sterile condition.

**CLINICIAN'S CHOICE<sup>®</sup>**  
DENTAL PRODUCTS INC.

London, ON, Canada, N6A 1M6 • Brookfield, CT, USA, 06804

1-800-265-3444 • (519) 641-3066

**CAUTION:** U.S. Federal law restricts this device to sale by, or on the order of, a dentist.



Emergo Europe  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands

122012  
987928