

INTENDED USE: Dental finishing and polishing points intended as a medical device for the purpose of final polishing of all composite hybrid and nanofilled restorations.

INSTRUCTIONS FOR USE

TO POLISH COMPOSITE: Using a slow speed handpiece between 8,000-15,000 RPM, apply light pressure with intermittent water spray in a wiping motion. In just 30-60 seconds, a high gloss finish will appear.

TO SHAPE & POLISH COMPOSITE: Using a slow speed handpiece at between 8,000-15,000 RPM, with water spray, apply moderate to heavy pressure in areas requiring re-shaping. Continuously check surfaces while contouring composites to achieve desired results. After contouring the composite, apply only light pressure with intermittent water spray until a high gloss finish is achieved.

PLEASE NOTE: Contouring composite will degrade Shape & Shine Polishers more quickly than polishing alone.

RECOMMENDED STERILIZATION

Sterilize after each use. Autoclave at 132-135°C/270-275°F. Product is not sold in sterile condition.

CLINICIAN'S CHOICE[®]
DENTAL PRODUCTS INC.

London, ON, Canada, N6A 1M6 • Brookfield, CT, USA, 06804
1-800-265-3444 • (519) 641-3066

CAUTION: U.S. Federal law restricts this device to sale by, or on the order of, a dentist.

EC

REP

Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands

102010
984105