

**INTENDED USE:** Dental finishing and polishing points intended as a medical device for the purpose of final polishing of all composite and ceramic restorations.

## **TECHNIQUE**

**STEP 1:** D-Fine Double Diamond Primary Polishers (purple)

1. Use a finishing diamond/carbide to contour the composite surface.
2. Select the appropriate D-Fine Double Diamond Primary Polisher.
3. Using a wiping motion, at 7,500 -10,000 rpm, and with light to moderate pressure and water spray, apply the primary polisher to the restorative surface. A smooth silky surface will appear in approximately 30 seconds.

**STEP 2:** D-Fine Double Diamond Final High Shine Polishers (peach)

1. Select the appropriate D-Fine Double Diamond Final High Shine Polisher.
2. Again, with light to moderate pressure and water spray, apply the Final High Shine Polisher to the restorative surface with a wiping motion, until a high gloss finish is achieved.

**NOTE:** Failure to follow this technique will lead to premature degradation of the polishers.

## **RECOMMENDED STERILIZATION**

D-Fine Double Diamond Polishers should be steam autoclaved only up to 275°F/135°C. Product is not sold in sterile condition.



**Clinician's  
Choice®**



Clinician's Choice Dental Products, Inc  
167 Central Ave., London, ON Canada N6A 1M6  
1.800.265.3444 • 519.641.3066  
[clinicianschoice.com](http://clinicianschoice.com)

**Rx** only

987928  
052022  
Made in  
Germany