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TempTray™

 Clinician's Choice Dental Products, Inc.
+1 519.641.3066 clinicianschoice.com

9000980434
122021

Instructions For Use

Indications for Use

Disposable aluminum tray for the fabrication of a temporary matrix.

Warning:

- This product must only be used by dental personnel.
- Please read instructions before use.

Shelf Life: 5 years

Directions for Use

1. Load the TempTray with a matrix material (Template® Ultra Quick Matrix Material) or other suitable impression material, following manufacturer's instructions for use.
2. Seat filled TempTray intraorally and allow the material to cure.
3. Remove the tray and proceed with provisional fabrication.

NOTE: TempTrays are not sold in sterile condition.

NOTICE: Any serious incident that has occurred in relation to the device should be reported immediately to Clinician's Choice Dental Products, Inc. and the competent authority of the member state in which the incident occurred.

Storage

TempTrays should be kept in its original packaging at room temperature and protected against dust, direct sunlight, moisture and corrosive gas until use.

Symbols Used

	Warning		Consult instructions for use
	Lot number		Use by date
	For single use		Catalogue number
	Manufacturer		EC authorized representative
	Keep away from sunlight		Keep dry
	Non returnable if seal is broken		CE marking

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