

**INTENDED USE:** Dental finishing and polishing points intended as a medical device for the purpose of final polishing on all composite hybrid and nano-filled restorations.

Shape & Shine Polishers have the ability to shape and contour cured composite material as well as create an unrivalled polish, all in a single step. Diamond-impregnated Shape & Shine Polishers can also be used to conservatively contour composite when additional pressure is applied. During polishing, the diamond particles continuously break down, getting smaller in just seconds, and provide a high luster finish to all hybrid and nano-filled composite materials.

**INSTRUCTIONS FOR USE:**

**To Polish Composite:** Using a slow speed handpiece between 8,000 -15,000 RPM, apply light pressure with intermittent water spray in a wiping motion. In just 30-60 seconds, a high gloss finish will appear.

**To Shape & Polish Composite:** Using a slow speed handpiece at between 8,000-15,000 RPM, with water spray, apply moderate to heavy pressure in areas requiring re-shaping. Continuously check surfaces while contouring composites to achieve desired results. After contouring the composite, apply only light pressure with intermittent water spray until a high gloss finish is achieved. PLEASE NOTE: Contouring composite will degrade Shape & Shine Polishers more quickly than polishing alone.

**RECOMMENDED STERILIZATION:** Sterilize after each use. Autoclave at 270-275°F/132-135°C.



Distributed by Clinician's Choice,  
a division of Den-Mat Holdings, LLC  
1017 W. Central Avenue, Lompoc, CA 93436 USA  
800.433.6628 [clinicianschoice.com](http://clinicianschoice.com)

**Rx** only



Made in Germany  
984104 R:1 07/24SN