

INTENDED USE: Dental finishing and polishing points intended as a medical device for the purpose of final polishing of all composite and ceramic restorations.

TECHNIQUE

STEP 1: D-Fine Double Diamond Primary Polishers (purple)

1. Use a finishing diamond/carbide to contour the composite surface.
2. Select the appropriate D-Fine Double Diamond Primary Polisher.
3. Using a wiping motion, at 7,500-10,000 rpm, and with light to moderate pressure and water spray, apply the primary polisher to the restorative surface. A smooth silky surface will appear in approximately 30 seconds.

STEP 2: D-Fine Double Diamond Final High Shine Polishers (peach)

1. Select the appropriate D-Fine Double Diamond Final High Shine Polisher.
2. Again, with light to moderate pressure and water spray, apply the Final High Shine Polisher to the restorative surface with a wiping motion, until a high gloss finish is achieved.

NOTE: Failure to follow this technique will lead to premature degradation of the polishers.

RECOMMENDED STERILIZATION

D-Fine Double Diamond Polishers should be steam autoclaved only up to 275°F/135°C. Product is not sold in sterile condition.



**Clinician's
Choice®**

Distributed by Clinician's Choice,
a division of Den-Mat Holdings, LLC
1017 W. Central Avenue, Lompoc, CA 93436 USA
800.433.6628 clinicianschoice.com

Rx only

Made in Germany
987928 R:1 06/24SN