Instructions for Use

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Mode d'emploi

L U M I N E E R S[®]

Starter Kit

BASIC STEP-BY-STEP

NOTE: PeriOptix[™] magnification recommended.

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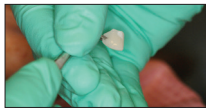
CAUTION: Wear protective gloves while using this product.

CAUTION: Wear eye protection while using this product.

CAUTION: This product has not been studied in children, pregnant or breast-feeding women.

PRELIMINARY RECOMMENDATIONS

NOTE: Day before appointment, dry try Lumineers on stone model to insure fit.

NOTE: Always remove **Ultra-Bond® Plus** from the refrigerator about one hour prior to use to allow time for the material to return to room temperature.

NOTE: It is imperative that you use magnification, such as the **PeriOptix®** line of magnification. You will be able to work more effectively and precisely. Your own prescription can be placed in the eyeglass lenses.

1. PREPARATION OF LUMINEERS® TO ENAMEL OR DENTIN SURFACES

- After the patient is seated, treat the Lumineers with **Porcelain Conditioner** for 30 seconds. Rinse and dry well (1a).
- Apply **Cerinate Prime®** to the Lumineers for 30 seconds and blow thin (1b).
- Place **LUMINEERS** back into the case box in their proper slots. Close lid (or cover from light). Once **Cerinate Prime** is applied to Lumineers, they need to be shielded from the light until the Lumineers are seated (1c).

2. TRY-IN OF LUMINEERS

- Apply the mixture of **Ultra-Bond Plus Try-In Paste** and **Cerinate Shade Modifier/Opaquer** into each of the Lumineers (2a).
- Gently place the Lumineers one at a time and use a brush to clean up the excess **Ultra-Bond Plus Try-In Paste** (2b).

NOTE: We recommend the **Lumi-Grip** to prevent dropping.

NOTE: To make shade and opacity modifications, we recommend the **Custom Shade Modification System II**, which contains 4 opaques and 12 modifiers. Mix **Ultra-Bond Plus Try-In Paste** and the **Shade Modifier/Opaquer** (no catalyst required) until well-blended and apply to the prepared Lumineers. Adjust the ratio of **Ultra-Bond Plus Try-In Paste** to **Shade Modifier/Opaquer** to achieve desired result. Make note of the final mixing ratio and use the same proportions for final cementation using **Ultra-Bond Plus** cement.

- Hold a mirror approximately 16 inches away from the patient's face and let the patient examine his/her smile (2c). If the shade is not what you or the patient wants, remove the **Ultra-Bond Plus Try-In Paste** from the Lumineers and tooth surfaces with a **Skubes®** sponge or a clean, dry brush. Reapply a new shade of **Ultra-Bond Plus Try-In Paste** and check the shade again.
- Remove Lumineers after try-in by using a **Schure 349** instrument or cotton pliers and gently tease the Lumineers off the tooth surfaces (2d).
- Remove excess **Ultra-Bond Plus Try-In Paste**. Place Lumineers back in the case box, with facial surfacing facing up, in their proper slots. Close lid (or cover from light).

NOTE: There is no need to reapply **Porcelain Conditioner** or **Cerinate Prime** if you are cementing directly after try-in.

3. PREPARATION OF TOOTH SURFACES

- Clean the teeth with a slurry of pumice and water or **Porcelain Laminate Polishing Paste** and floss (3a).
- Paint-On Dental Dam** (optional) is helpful and recommended for Lumineers placement. Before bonding, syringe **Paint-On Dental Dam** on the lingual side of the interproximals, covering gums and teeth thoroughly. Light-cure for 5 seconds per tip width with the **Sapphire®** Plasma Arc Curing (PAC) light. This will make your clean-up faster and easier (3b).
- Etch teeth for 20 seconds with **Etch 'N' Seal®**. Rinse and dry (3c).
- Apply dental adhesive of your choice per manufacturer's instructions, however, ensure final curing of Ultra-bond cement by curing light.

4. BONDING LUMINEERS® TO AN ENAMEL OR DENTIN SURFACE

NOTE: If you are bonding Lumineers to a porcelain surface.

- Select the appropriate shade of **Ultra-Bond® Plus** resin cement (4a).

NOTE: Prior to placing the auto-mixer tip on the **Ultra-Bond Plus** syringe, always extrude a 1/4-inch line of material before using to ensure adequate mixing of the material.

NOTE: **Ultra-Bond Plus** has an extended working and set time to permit placement of multiple laminates. We suggest that the clinician dispense a small amount of **Ultra-Bond Plus** onto a mixing pad a minute or two before beginning placement and monitor it for working time to provide warning in the event that the placement takes longer than anticipated.

NOTE: If patient has dark stains, such as tetracycline stains, we recommend using the **Cerinate Shade Modification System II**. If shade modification or basic opaque adjustment is necessary, refer back to the SHADE ADJUSTMENT section. If not, proceed to the next step.

- Apply **Ultra-Bond Plus** to the prepared Lumineers and put in place (4b).

NOTE: We recommend using the **LUMIgrip®** and holding over patient's bib to avoid dropping.

- After placing Lumineers, remove excess **Ultra-Bond Plus** (4c).
- Tack Lumineers in place for 2 seconds using the **Sapphire® PAC** light fitted with a **Ceri-Taper®** 2 mm tacking tip (see Fig. 2). Do not touch our press on Lumineers while curing (4d).
 - Place and tack cure centrals 8 and 9.
 - Place and tack cure cuspids 7 and 10.
 - Place and tack cure bicuspids 6 and 11, then tack cure 5 and 12.

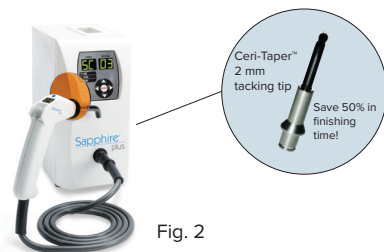
Fig. 2

VERY IMPORTANT TOOL!

- Continue to remove excess cement with a soft brush.
- Using a 9 mm tip and the **Sapphire PAC** light, completely cure all Lumineers, exposing each surface for 5 seconds (4f).
- Remove paint-on dam if placed.

5. FINISHING

At the placement appointment, only minimal finishing is suggested to finish the DEJ and to remove any excess resin cement. Do not open the embrasures. Maximum bonding occurs in 24 hours. Final finishing should be performed at next visit. We recommend the Lumineers Finishing Kit which includes all components listed in the following instructions. (See Fig. 3 to reference the burs used during finishing).

- Use a **Schure 349** instrument to remove the cured cement along the margins, as well as from the interproximal area along both the facial and the lingual. You may also use the **12 Fluted Carbide Bur (C)**. This bur is an excellent tool for cleaning up residual resin cement and for finishing composites.
- Clean up any leftover **Ultra-Bond® Plus** using the **12 Fluted Carbide Bur (C)**.
- Using a **Mosquito Interproximal Diamond (F)**, remove the porcelain ledge, if any. Using the **Mosquito Interproximal Diamond (F)**, refine the margins along the interproximal area, if necessary. If necessary, further polish the margins using the **Mosquito Interproximal Diamond (F)** again.
- Check the bite with articulating paper. Use the **Football Diamond (E)** on the lingual to finish the incisal wrap and adjust high spots or length.

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- Trim/polish the margins of the porcelain veneer, if necessary. Use a **Extra-Fine Smoothing Diamond (B)** to smooth out porcelain.
- Explore the tooth structure next to the Lumineers to check margins. There should not be a catch (or bump).
- Use the **CeriSaw** to separate the teeth using a rocking motion.
- Use the **CeriSander** (fine grit) to smooth the interproximal surfaces.
- Use the **CeriSander** (ultra-fine grit) to further smooth the interproximal surfaces. It is important that you use each sanding blade only two or three times per tooth to smooth contacts.
- Check the interproximal surfaces with dental floss for smoothness.
- Continue with **Porcelain Laminate Polishing Paste** using the **Polishing Cup (G)**—use side of cup and assistant follows doctor with high speed evacuation tip on.

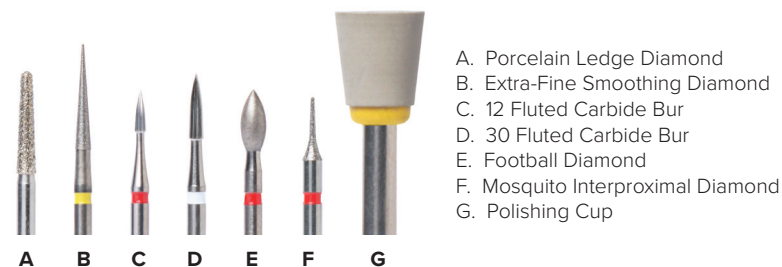
Fig. 3

Lumineers Starter Kit

All The Tools You Need for Lumineers Placement

This system eliminates a cabinet of four to six different kits for your lumineers placement. No more trying to remember everything you need to have ready at chairside. In a convenient, ergonomically designed package, this system takes you through impression, try-in, preparation, placement and finishing. Each tray is designed to fit your chairside tray.

If you have any questions, please contact us at international@denmat.com today or visit denmat.com

LUMINEERS STARTER KIT – EVERYTHING YOU NEED IN ONE KIT!

	CERINATE PRIME® & CONDITIONER Bonding agent and etchant to bond porcelain to natural tooth structure.
	PAINT-ON DENTAL DAM Paint-On Dental Dam, a light-cure resin paste, will help protect gingival tissue from chemical agents. Ideal for use prior to bonding. Syringe Paint-On Dental Dam on the lingual side of gums and teeth. This will make clean-up faster.
	ULTRA-BOND® PLUS TRY-IN PASTE Lumineers® made of Cerinate® Porcelain, if placed on the model, is intended to have a loose fit to allow room for the restorative, Ultra-Bond® Plus, an ADA Type II resin cement. Ultra-Bond Plus Try-In Paste is formulated to match the shade of the cured Ultra-Bond. Use to try-in restorations to verify shade.
	ETCH 'N' SEAL® Etch 'N' Seal combines a 25% phosphoric acid with 0.5% aluminum oxalate so you can etch enamel and condition dentin at the same time. The combined use of Etch 'N' Seal and the Tenure Multi-Purpose Bonding System results in the formation of a dentin hybrid layer that can reduce dentin permeability, protect against acid attack and decrease sensitivity.
	PORCELOCK® PorceLock is the perfect companion for porcelain, and crown-and-bridge repair. For repair of porcelain surfaces, the use of PorceLock and Cerinate Prime® with Ultra-Bond allows you to achieve strong and reliable intraoral bonds. PorceLock's gel viscosity and needle tip dispensing allow you to place the gel only where you want it. PorceLock is a buffered intraoral and extraoral use hydrofluoric acid gel that is both safe and effective. Its formula contains 2.5% hydrofluoric acid. PorceLock creates bond strengths three to five times greater than treatments using silane alone. It achieves bond strengths approaching those of etched enamel after a 3–5 minute treatment.
	ULTRA-BOND® PLUS RESTORATIVE RESIN CEMENT Ultra-Bond is an ADA Type II Restorative proven to prevent cracking/crazing and pop-offs. Lumineers®, bonded with Ultra-Bond and Tenure®, creates a wear resistant, non-microleaking, highly esthetic restoration that is clinically proven in on-going studies to last over 20 years. Available in Syringeable or Powder-Liquid formulations.
	LUMINEERS FINISHING KIT Tools that make dentistry faster, easier, and better! This all-inclusive finishing system is the only one of its kind available to the dental profession. These simple tools save you time and are more comforting to the patient.

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