



## Porcelain Polishing Paste



REF 4021310

### CONTENTS:

- Porcelain Polishing Paste syringe, 3 pc. @ 15 g ea. 
- Syringe tips 18 G, 10 pc. **(NM-D)**
- All Surface Bonding Chart **(NM-D)**
- Instruction card **(NM-D)**

**Store in Cool Dry Location**



MDSS GmbH  
Schiffgraben 41  
30175 Hannover, Germany

**Symbols Glossary available at:**  
[denmat.com/symbols](http://denmat.com/symbols)



Den-Mat Holdings, LLC  
1017 W. Central Avenue  
Lompoc, CA 93436 USA

©2018 Den-Mat Holdings, LLC. All rights reserved.  
754023130 R6 11/18CM

Porcelain Polishing Paste is a non-aqueous, fine-particle, pleasant-tasting paste that has a smooth-flowing consistency, providing the finest luster on composite, as well as porcelain restorations. Porcelain Polishing Paste produces superior polishing results and reduces material waste.

**INSTRUCTIONS**

- 1. Remove the syringe cap (do not discard) and attach the syringe tip.
- 2. Dispense directly onto tooth surface and polishing point or cup.  
Polish in a circular motion and refresh as needed.
- 3. Rinse the polished surface thoroughly.
- 4. Remove the syringe tip and replace the syringe cap.

**STORAGE**

Store in a cool, dry location  
Do not expose to temperatures exceeding 77° F (25° C).  
Do not freeze  
Cold storage is not required

**LIMITED WARRANTY**

DenMat will replace, or refund the purchase price of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price shall be the exclusive remedy for the user. DenMat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.

**DENMAT CUSTOMER SUPPORT**

|                                               |              |
|-----------------------------------------------|--------------|
| U.S. and Canada Customer Service.....         | 800-433-6628 |
| U.S. and Canada Fax Orders and Inquiries..... | 800-233-6628 |
| International Inquiries .....                 | 805-347-7990 |
| International Fax Orders and Inquiries .....  | 805-347-7983 |
| Website .....                                 | denmat.com   |

**SAFETY DATA SHEETS AVAILABLE AT [denmat.com](https://denmat.com)**