

MegaMix Cartridges

INSTRUCTIONS FOR USE

Preparation of the Cartridge

Hold the cartridge with the sealed outlet openings facing upwards. Take the handle of the sealing cap and remove the sealing cap upwards. Avoid excessive force.

Insert the cartridge into the mixing machine (according to manufacturer's instructions). Attach a new mixing tip to the cartridge. After the mixing tip is correctly seated (the central hexagonal socket at the lower end of the tip must be properly aligned with the hexagonal drift shaft), place the Bayonet ring over the mixing tip and push it down completely. Turn the ring clockwise (1/4 turn) to lock the mixing tip securely in place.

Check to ensure the hexagonal drift shaft engages when starting. The spiral in the mixing tip must rotate when starting.

NOTE: Discard the first 3cm of impression material extruded from newly filled cartridges. Store used cartridge with fixed/used mixing tip in place. It serves as a cap.

To remove the used mixing tip turn the Bayonet ring counter-clockwise and draw it off.



Clinician's Choice®

 Clinician's Choice, a division of Den-Mat Holdings, LLC
1017 W. Central Avenue, Lompoc, CA 93436 USA
800.433.6628 clinicianschoice.com