

120mL PowerMix™ cartridges

INSTRUCTIONS FOR USE

For complete instructions and further detail on specific operating elements, please refer to the PowerMix Operating Guide.

Inserting the Cartridge

- Make sure the mixing tip is NOT attached. Lift the cartridge release lever on the PowerMix Dispenser using index and middle fingers.
- Make sure the plunger indicator is all the way back at the start position.
- Remove cartridge cap and dispose.
- Insert the cartridge into the cartridge bed.
- Push the cartridge lock down with your fingers.
- Attach a new mixing tip prior to mixing.
- Push the plunger indicator manually forward until it is stopped by the back of the cartridge.

The mixing process will not start until the plungers are touching the back of the cartridge.

Attaching the Mixing Tip

- Pull the locking ring upwards until it clicks.
- Line up the metal mixing shaft of PowerMix with the corresponding part (in black) of the mixing tip. Push in lightly.
- Turn the mixing tip slightly and continue pushing until the 2 circular recesses on the back of the mixing tip line up with the corresponding outflow openings of the cartridge. Push in firmly until it bottoms out.
- Press down the locking ring firmly to ensure that the mixing tip and the cartridge are fully engaged.

NOTE: The locking ring cannot be pushed down fully until the mixing tip and cartridge are fully engaged.

CAUTION: The mixing tip locking ring must be in the **DOWN** and **LOCKED** position before turning the machine **ON**. Failure to do so will result in 5 beeping sounds and unit will not work.

Always attach a **NEW** mixing tip and ensure that it is fully seated before pushing the 'Start' Button.

After mixing, leave the mixing tip on the cartridge as a seal. Do not remove the mixing tip before the impression material fully sets.



Clinician's Choice®

 Clinician's Choice, a division of Den-Mat Holdings, LLC
1017 W. Central Avenue, Lompoc, CA 93436 USA
800.433.6628 clinicianschoice.com

981170 R:2 08/25CM